

All your Embryo Transfer Flush Media and Plastics Supplies

- EMCARE™ COMPLETE Ultra flushing media contains ICPbio low endotoxin bovine albumin providing surfactant and embryo protective properties, with binding sites absorbing plasticizers, endotoxins and proteases.
- Low foaming formulation - 50% less foaming than previous formulations.

Flushing Media Catalogue	Units	Cat. No	
COMPLETE Ultra Embryo Flushing Solution	1L	CECFS-100	
COMPLETE Ultra Embryo Flushing Solution	2L	CECFS-200	
EMCARE™ Flushing Solution (without Albumin)	1L	ECFS-100	
EMCARE™ Flushing Solution (without Albumin)	2L	ECFS-200	
20% Albumin™ Concentrate	5X20ml	ABRD-100	
Handling Media Catalogue	Pack	Units	Cat No
EMCARE™ Embryo Holding Solution	Minipac	20x6ml	MPHM-100
	Bottle Pack	5x20ml	ECHM-100
	Bottle Pack	5x100ml	ECHM-500
with 10% Glycerol	Minipac	20x6ml	MPGF-100
	Bottle Pack	5x20ml	ECGF-100
with 34.2% Sucrose (1M)	Minipac	20x6ml	MPST-100
	Bottle Pack	5x20ml	ECST-100
with 1.5M Ethylene Glycol	Minipac	20x6ml	MPEG-100
	Bottle Pack	5x20ml	ECEG-100
EMCARE™ Embryo Thawing System	Minipac	4x4x6ml	MPTP-80
	Bottle Pack	4x20ml	ECTP-80

*Embryo Thawing System

Thaw



1. 6% v/v glycerol + 0.3 (10.3% w/v) sucrose in EMCARE™
2. 3% v/v glycerol + 0.3 (10.3% w/v) sucrose in EMCARE™
3. 0.3M (10.3%) sucrose in EMCARE™
4. EMCARE™ hold

* To use Embryo Thawing System: Thaw straw 15-20 seconds in 37°C water bath then pass embryos through each of the 4 solutions 6-7 minutes each.

Contains 0.4% ICPBioReproduction bovine albumin

Plastics

Part Number	Product Description	Pack
04197	Bardia Foley Cath. 18" 18FR 30cc	1 unit
04135	VCI Em Con Filter, 75 micron stainless steel mesh, sterile	1 unit
04136	VCI High Volume E.T. Filter, 120ml, 65 micron nylon mesh, sterile	1 unit
04137	VCI Low Volume E.T. Filter, 50ml, 65 micron nylon mesh, sterile	1 unit
23138	VCI Sureflush Filter, 75 micron nylon mesh, sterile	1 unit
412012	Mini Spike filter	1 unit
04045	Dish 4 well ESP 10/pkg	10/pkg
03-500-125-1	Syringe flush system foley 3-part	1 unit
03-301-192-2	Y junction tubing long w/rousch connector	1 unit
04132	Y junction tubing long w/foley connector	1 unit
03-510-125-1	Extension Kit w/LL Adapter	1 unit
03-530-125-1	Valve 3 way, disposable and sterile	1 unit
04814	IMV Transfer Sheaths Heifer (blue)	5/pack
04169	IMV Transfer Sheaths Cow (clear)	5/pack
04140	Disposable Implant Gun, 1/4cc, 21" (25/pack)	25/pack
04141	Disposable Implant Gun, 1/2cc, 21" (25/pack)	25/pack
04808	IMV 21" plastic roll D801 Chemise*	70/roll
04810	IMV 18" plastic roll D800 Chemise	100/roll
19532	Ultra-Sens™ Sleeve - individually packed and gamma irradiated	25/pack
04020	Petri dish round with grid	20/pack
04917	Cane Tabs 3xyl 1 x wh	100/pack
04166	Straw 1/4cc sterile	5 pack
5/119/79/4	Straw plug 1/4cc IMV	100/pack
04910	Goblet 10 mm (small)	1 unit
04912	Goblet 13 mm (large)	1 unit
04915	Cane alumi 10 mm (small)	1 unit
04916	Cane alumi 13 mm (large)	1 unit
18485	Cito Thaw Unit, Polypropylene 230 v AC - 2 yr warranty	1 unit
18490	DM Digital Stainless Steel Semen Thaw Unit, 230v AC - 2 yr warranty	1 unit

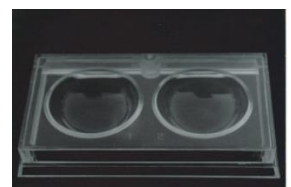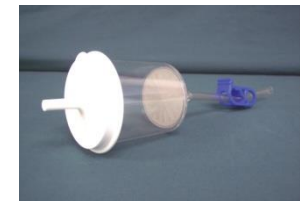

Dairymac Limited

Lakeview, Unit C Waltham Business Park, Brickyard Road, Swanmore, Hampshire SO32 2SA, England

Tel/Fax: +44 (0)1489 894447 • E-mail: sales@dairymac.com

www.dairymac.com • www.estrotect.com