

## Instructions for Application


### Keep Heat Detectors Warm

Heat activates the adhesive. Store at or above human body temperature when you're about to apply. Warm to 100°F or 38°C immediately prior to application.



### Site of Application

Ideal location for placement is between hip and tail head, perpendicular to the spine.

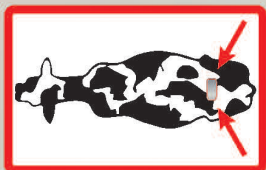

### Brush Hair Thoroughly

Brush with the grain of the hair to remove dirt, dust and shedding or molting hair.



### Clean the Hair

Clean the area with an ESTROTECT Cleaning Cloth to completely remove any remaining dirt, dust or sand.



### Heat Detector Application

Place detector on cow, press down firmly and rub into hair with bare fingers.



**ESTROTECT™**  
HEAT DETECTOR

U.S. Pat. #6,467,430