



INSTRUCTIONS FOR CORRECT USE AND CARE OF YOUR *THRIFTY150™ DIP CUP

The THRIFTY150 [™] can reduce chemical usage by over 75% when used correctly

Instructions for first use

- 1. Unscrew the top from the bottle and fill with fresh dip to just below neckline
- 2. Screw top back on until tight
- 3. Squeeze the bottle to wet the brushes there is no need to fill the chamber but a small amount of chemical may form in the bottom of the dip cup chamber
- 4. Each gentle squeeze between cows wets the brushes with sufficient dip for 4 teats



Fill Level

THD/001 Post Dip





- 5. There are two versions of the THRIFTY150 a blue head with small channels for thinner very fluid dips often used for pre dipping and a clear head for normal, thicker dips and post dipping procedures. The Post Dip device is much easier to squeeze. The same brush unit is used on both.
- 6. It takes about a week for the brushes to 'break in' and 'soften up' and to give consistent savings.

Dipping Action

- 1. Gently squeeze the cup to thoroughly wet brushes
- 2. Immerse each teat fully, small teats should go through the hole in the brushes, a slight twisting action helps but may take a little practice and is easier once the brushes break in.
- 3. Withdraw the Dip Cup slowly to ensure good coverage and maximise savings!
- 4. It takes a little time to develop a precise dipping action but the savings and coverage are worth it- after a few days famers report that they dip just as quickly but spend less time filling Dip Cups.

Cleaning

- 1. To prevent contamination of brushes it is good practice to clean them every few days.
- N.B. Cleaning is recommended for all Dip Cups.
- 2. Methods:
 - Soaking the top unit over night in a weak solution of mild bleach and water this allows the brushes to relax and unfurl if tangled
 - Gently massaging brushes with soapy water and fingers
 - Do not use harsh parlour detergents as these will shorten the life of the brushes

Removal of Brushes and Inner Bowl

- Remove the retaining ring and pop the brushes out, the inner bowl will also pop out but may require insertion 1. of a small screwdriver to lever out.
- Reassemble in reverse order. 2.
- 3. Spare brushes may be purchased from your local dealer.

Part numbers for replacement parts

- THD/001 THRIFTY150 Post Dipper complete (Clear)
- THD/002 THRIFTY150 Pre Dipper complete (Blue)
- THD/003 Red Replacement Brush
- ECD/005 Bottle (500ml) Replacement
- ECD/008 Top Cup (no brush) Replacement

HELPLINE Number +44 7872 161128

*THRIFTY150 is a is a trademark of Mastitis Management Inc Dairymac Limited

Brook House 3 Brookview, Tanfield Lane, Wickham, Hampshire, PO17 5NW, UK Tel: +44 (0) 1329 835571 Fax: +44 (0) 1329 835571 Email: sales@dairymac.com : Web: www.dairymac.com Company Number: NI064477 VAT Number: GB 909 2822 16

