

StopLik™

Lick Deterrent Aid



KICK THE LICK HABIT WITH STOPLIK

StopLik Helps Pets Conquer Habitual Licking

Now pets, their owners and veterinarians can get fast-acting relief from habitual licking problems with StopLik.

New Option to An Old Problem

StopLik deters the habitual licking that can cause open sores.

It also discourages pets from removing bandages during healing, allowing the open sores to be more successfully treated.

Designed to discourage pets from licking a sore or wound, StopLik is applied over a bandage placed over the site and delivers a mild electric stimulus to the tongue of the pet when licked. Harmless to the pet – the stimulus is similar in strength to one from a watch battery – StopLik is activated by moisture. Pets who feel the tingling on their tongue and the unpleasant taste eventually stop licking.

In trials at veterinary clinics, StopLik has worked on a high percentage of all sizes and breeds of dogs.

Veterinarians are also using StopLik to discourage pets from pulling out IVs, Seton drains and urinary catheters or licking surgical stitches on their abdomen and torso.

Stoplik Stops The Urge to Lick

- ✓ Unique, simple
- ✓ No drugs
- ✓ Highly effective, fast-acting relief for most pets
- ✓ Accepted by pets and owners
- ✓ Field-tested
- ✓ Non-hazardous
- ✓ Alternative to Elizabethan Collar, Bitter Apple Spray
- ✓ Color-coded

EFFECTIVE TREATMENT FOR:

- ✓ Lick Granuloma
- ✓ Licking, tearing at surgical stitches
- ✓ Pulling out IVs, Seton drains, urinary catheters
- ✓ Dogs, cats or other small animals

StopLik is offered in color-coded 1.62 and 3 inch widths and two versions for moderate to strong temperaments. Each easy to use dispenser holds 12 devices. Each lick deterrent aid is individually packaged with directions for application.

Color	Width	Lick Habit
Green	1.62 inch	Moderate
Blue	3 inch	Moderate
Purple	1.62 inch	Strong
Orange	3 inch	Strong

Kick The Lick Habit

1.62M 3M 1.62S 3S

StopLik™

Lick Deterrent Aid

Kick The Lick Habit

APPLYING StopLik Is Simple

To activate StopLik:

Remove brown paper backing from the front of the device on the battery end.

Fold end over, so the B symbol is placed onto the top of the battery. The target holes should line up to each other. Press material firmly onto and around the battery to ensure activation.

For leg applications:

Remove the white backing material from the StopLik.

Apply StopLik over the bandage starting with the battery end, wrapping around the bandage and back over itself, just past the battery, allowing a 1 inch overlap. Trim off excess material with scissors so the breathe holes are not sealed off.

For other than leg applications:

Before applying to the bandage, fold battery end back onto the adhesive on the backside of StopLik. Now StopLik is ready to be applied to the bandage area.

StopLik can be cut to length from the opposite end of the batteries. Trimming does not change the effectiveness of the device.

StopLik can be applied to stop the animals from removing IVs, urine catheters, Seton drains or pulling at bandaged abdominal surgical sites.

See how StopLik is applied and works by visiting www.stoplik.com and viewing an educational DVD on StopLik application.

StopLik™





Lick Deterrent Aid

Kick The Lick Habit



Dairymac Limited
Brook House, 3 Brookview
Tanfield Lane, Wickham
Hampshire, PO17 5NW
Tel/Fax : + 44 (0) 1329 835571
Email : sales@dairymac.com
Web : www.dairymac.com

Dr. Joshua W. Stewart,
Wales Animal Clinic, WI
and StopLik patient.

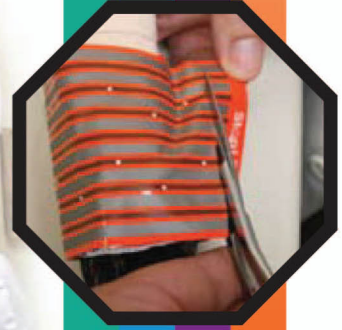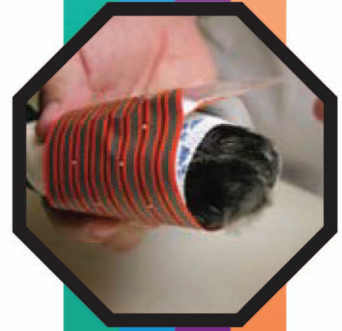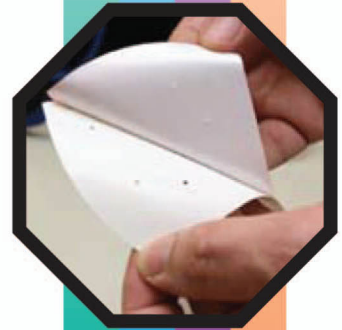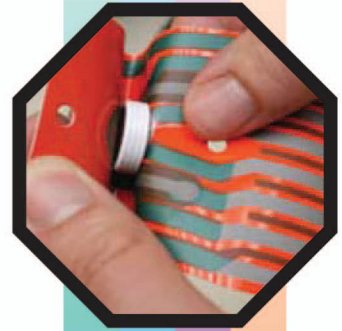

Kick The Lick Habit

1.6M 3.0M 1.6S 3.0S